

Pro-Cleanse™

Instrument Disinfectant



Pro-Cleanse™



Pro-Cleanse Instrument Disinfectant provides a safe infection control solution for instruments and equipment. Pro-Cleanse Instrument Disinfectant is a routine disinfectant and can be used with any heat sensitive equipment. For best results use in conjunction with Pro-Cleanse triple Enzymatic Cleanser.

HIGH LEVEL INSTRUMENT DISINFECTANT

Pro-Cleanse™

Pro-Cleanse™

Instrument Disinfectant



Pro-Cleanse™

Product Description

Pro-Cleanse High Level Instrument Disinfectant is a highly effective sanitiser for cleaning and disinfecting all surgical instruments. Use as part of your daily cleaning and infection control routine.

Applications

- Broad spectrum biocide, highly effective against bacteria, viruses, fungi and yeasts.
- Cleans and protects without abrasion, bleaching, odour or taint
- Suitable for use on all materials including metal, glass, plastics, rubber & polymers

Anti-Microbial Efficacy

High Level Instrument Disinfectant passes European tests EN13704, BS EN14204, BS EN1276, BS EN14348) for effectiveness at use of dilution against MRSA, Salmonella, Hepatitis B, TB (Mycobacterium tuberculosis), Bacillus (BS Cereus, Clostridium Perfringens and Black Mold (Aspergillus Niger).

Directions for Use

Pre-cleaning:

First clean the instruments using Pro-Cleanse Enzymatic Cleanser and rinse thoroughly before soaking with Pro-Cleanse Instrument Disinfectant.

Cleaning/Sanitisation:

Use at a dilution of 1:100 for high level disinfection of instruments. Soak instruments in solution for 10 minutes and rinse thoroughly using purified water.

Precautions

Ensure product is diluted according to guidelines given above. DO NOT breathe in fumes/vapours/spray. Wear protective gloves and clothing whilst handling. DO NOT swallow or inhale. If contact is made with skin or eyes rinse cautiously with water for several minutes.

Physical Description

Potent blend of highly active quaternary ammonium compounds & non-ionic surfactants

Appearance	Opacity	Colour	Density	pH
liquid	partially clear	pink	1.00g/ml	10.5-11.5

Pro-Cleanse™

HIGH LEVEL INSTRUMENT DISINFECTANT